

ONE TOUCH Meter

HIGH RANGE = 240-391
LOW RANGE = 41-65

ONE TOUCH Test Strip

Serial # _____

Lot #: _____ Expiration Date: _____

Check Strip Range = Range on the monitor

Code #: _____

Solution Expiration Date (High): _____ Solution Lot: _____

LifeScan Customer Service: 1-600-227-8862

Solution Expiration Date (Low): _____ Solution Lot: _____

MONTH: _____

If Center Is Not Open - Write CLOSED

1ST WEEK	DATE	TIME	OPERATOR INITIALS	CONTROL SOLUTION READING HIGH	CONTROL SOLUTION READING LOW	CHECK STRIP	METER CLEANED (✓)	COMMENTS
Monday								
Tuesday								
Wednesday								
Thursday								
Friday								
2ND WEEK								
Monday								
Tuesday								
Wednesday								
Thursday								
Friday								
3RD WEEK								
Monday								
Tuesday								
Wednesday								
Thursday								
Friday								
4TH WEEK								
Monday								
Tuesday								
Wednesday								
Thursday								
Friday								
5TH WEEK								
Monday								
Tuesday								
Wednesday								
Thursday								
Friday								

• Calibrate Qday with ✓ strip.

* Clean Qwk and PRN using a cotton ball with (warm) soap and water. After cleaning, use solution to check for proper functioning of machine then calibrate with ✓ strip

Operator Signature: _____ Initials: _____

Operator Signature: _____ Initials: _____

Operator Signature: _____ Initials: _____